



**Onyfix®**

**FAQ**

**ONYLUX LIGHT - GENERAL USE**

en

**TIPS**

- (1) If you are finding that the Primer or composites are not correctly setting, or if your strips are falling off after only a few days, then you will probably have an Onylux light that is not fully charged.
- (2) Don't wait until your Onylux light is discharged prior to re-charging. Lithium-ion batteries actually prefer partial drainage to complete drainage. The Onylux battery should last about 6-8 hours' worth of run-time before it is completely drained.
- (3) Charge your Onylux light every 10-15 patients.
- (4) Don't overcharge/keep the battery charging after it has been completely charged (3 hours).
- (5) A tip is to recharge in the morning, then disconnect around lunchtime.
- (6) The battery should last approximately 300 recharge cycles.
- (7) When not in use store your light at room temperature, or ideally, in an air conditioned spaced. Heat is bad for the battery and will decrease its lifespan. Refrigeration is unnecessary.
- (8) If you are not going to be using your Onyfix for a while, we recommend taking the battery out of the Onylux light entirely. We recommend placing the battery in a cool area, away from direct sunlight or metal objects.

