



Onyfix®

FAQ

HOW TO SETUP ONYLUX LIGHT

en

STEPS

- (1) UNPACK THE BATTERY, LID AND CORD.
- (2) UNSCREW THE BATTERY HOLDER FROM THE BACK OF THE LIGHT.
- (3) PLACE BATTERY IN THE BATTERY HOLDER. MAKE SURE THE NEGATIVE END (-) IS INSERTED INTO THE LIGHT SHAFT, WITH THE POSITIVE (+) END ATTACHED TO THE BATTERY HOLDER.
- (4) SCREW THE BATTERY AND BATTERY HOLDER BACK IN THE LIGHT.
- (5) PRESS THE LID FIRMLY ON THE LIGHT.
- (6) PLUG IN THE INCLUDED USB CABLE AND CHARGE FOR 3 HOURS.

REPLACEMENT PARTS

Replacement parts for your Onylux® Toolkit can be purchased from PodCAM Supplies.

REQUIRE MORE ASSISTANCE?

Please email info@podcam.supplies if you require additional assistance setting up any part of your Onylux® Toolkit.

